



Hygiene Plan

Daily

After/before each patient

Hands
disinfect, wash if necessary

Additions:	All staff whose hands could be contaminated
------------	---

Instruments
disinfect all contaminated instruments and tools, rinse, package, sterilise (if necessary), and store in their appropriate place

Additions:	Staff member in charge:
------------	-------------------------

Wipe disinfection of the treatment unit, furniture, and all fixed apparatuses within a radius of 1.5 m from the treatment chair for example:

- swivel table
- treatment chair (head support, armrests)
- handpieces and contra-angles, turbine, multi-functional syringes
- handles of unit, operating light, and all used drawers
- operating keys (unit, chair)
- spittoon bowl
- connector pieces of dental pump, suction tube, saliva ejector
- X-ray tube (handle, trigger), if in close proximity to the patient
- all used small appliances, e.g. polymerisation lamp and other objects in close proximity to the patient

Additions:	Staff member in charge:
------------	-------------------------

Aspirating unit
rinse (aspirate water)

Working surfaces in the surgery
disinfect

Waste
dispose

Additions:	Staff member in charge:
------------	-------------------------

Daily

After consultation hours

Aspirating unit
clean (aspirate 2 litres of cleaning solution)

All surfaces, washbasins, fittings, floors in surgery and sterilisation room
clean and disinfect

Additions:	Staff member in charge:
------------	-------------------------

Weekly

After consultation hours

Aspirating unit
disinfect (aspirate 2 litres of disinfecting solution)

Working surfaces, washbasins, fittings and floors in the waiting room, reception, and bathroom as well as surfaces such as

- door handles
- telephone receiver, intercom system
- sanitary facilities

disinfect and clean

RX- film developer
disinfect outside

Instrument baths (incl. ultrasonic and impression bath)
change solution (a daily change is recommended)

Additions:	Staff member in charge:
------------	-------------------------

Periodically

Monthly (at least once or twice)

Autoclave
check performance in accordance with local regulations

Frequently used drawers
clean and disinfect inside

Every three months

Refrigerator, inside and outside
disinfect (more frequently, if necessary)

Every six months

Instruments in sterile packages
wrap in new sterile packaging, date and sterilise

Additions:	Staff member in charge:
------------	-------------------------

Safety clothing

- Wear disposable gloves for every treatment session
- Wear household gloves in the infectious (red) zone of the sterilisation room
- Wear mouth and eye protection when working in an environment where aerosols are released
- Change safety clothes daily!

Additions:	
------------	--

Waste treatment

- Place sharp and fragile objects in closable, leakproof, and puncture-resistant containers and dispose with surgery waste
- Place waste containing amalgam in closed, solid containers with odour trap
- Dispose X-ray and photo chemicals in special containers (observe local regulations)

Additions:	
------------	--

Hygiene Supervisor:	Toxicological Institute:
	Hospital:
	Other:

Distributor:	
--------------	--

Instructions for use



HANDS

Care, disinfection

1. Wash with liquid soap at the beginning and the end of the work day as well as before and after breaks or when contaminated
2. Dry with disposable towels
3. Disinfect before and after each treatment, before and after putting on the examination gloves, when leaving the operating theatre, or as required
4. Apply hand cream sparsely several times a day

Additions:	
------------	--



Recommended products

- OROLIN® Lotion
- OROMED® Gel
- OROSEPT® Solution
- OROCLEAN® Gloves
- OROCREME



SURFACES

Small surfaces
(e.g. swivel table, small appliances, operating elements, door handles)

Spray and wipe disinfection - make sure that all areas are completely moistened

Recommended products

- ISORAPID® Spray
- ISORAPID® Liquid
- DENTIRO® of your choice
- DENTIRO® Wipes
- ISORAPID® XXL Wipes

Floors, large surfaces, sanitary facilities

Wipe and mop disinfection - make sure that all areas are completely moistened

- OROLIN® OP-Forte AF
- ISORAPID® Floor

Additions:	
------------	--



INSTRUMENTS & TOOLS

Instruments
Disinfection, sterilisation

1. Immersion bath or washer-disinfector immediately after use
2. Rinse with water, check for cleanliness (clean again, if necessary), dry, oil if required, inspect for functionality and integrity
3. Place extraction forceps and other surgical instruments in sterile packaging and date
4. Sterilise in the autoclave (thermoresistant instruments only)

Recommended products

- OROCID-MULTISEPT® Plus
- OROLIN® Perasept

Burs, rubber polishing cups, brushes, finishers

1. Immersion bath or washer-disinfector immediately after use
2. If required, remove residues mechanically and repeat disinfection
3. Do not rinse drills. Place on paper cloth and allow to dry
4. Rinse other parts with water, check for cleanliness, dry, sterilize (if necessary), store

OROLIN® Burbath

Dental tools
(e.g. technical forceps)

1. Wipe disinfection
2. In case of contamination with saliva or blood, disinfect in the immersion bath or washer-disinfector like instruments
3. Sterilise only if thermoresistant
4. If required, use maintenance oil on instruments forceps

- ISORAPID® Spray
- ISORAPID® Liquid
- DENTIRO® of your choice
- DENTIRO® Wipes
- ISORAPID® XXL Wipes

Bath change

Replace working solutions weekly or if contaminated. Replace disinfectants based on peracetic acid daily

Additions:	
------------	--



ASPIRATING UNITS & IMPRESSIONS

Aspirating unit

Rinse with copious amounts of water after each patient
Aspirate cleanser at the end of consultation hours. Disinfect at least once or twice a week

Recommended products

- ORO CLEAN® Liquid
- ORO CLEAN® Plus

Impressions

Disinfection and cleaning in the immersion bath

ASEPTOPRINT® Liquid

Spray disinfection

ASEPTOPRINT® Spray

Additions:	
------------	--